

Comparison of Hi-Rem Post vs conventional posts

|                                                       | Hi-Rem Post | Conventional Posts |
|-------------------------------------------------------|-------------|--------------------|
| <b>Post properties</b>                                |             |                    |
| Soft cored to allow for easy removal whenever desired | yes         | no                 |
| Immediately identifiable as an "Hi-Rem Post"          | yes         | no                 |
| <b>Removal procedure</b>                              |             |                    |
| Very easy detection of the post centre                | yes         | no                 |
| No need of radiographs during removal procedure       | yes         | no                 |
| Trouble-free and fast operative procedure             | yes         | no                 |
| Easy-to-perform removal technique                     | yes         | no                 |
| Preservation of tooth tissues                         | yes         | no                 |
| No risk of creating perforations                      | yes         | no                 |
| No risk of canal enlargement                          | yes         | no                 |
| Predictable result                                    | yes         | no                 |
| Removal technique performed using standard burs       | yes         | no                 |
| No x-rays to the patient                              | yes         | no                 |
| Complete and safe post and cement removal             | yes         | no                 |

Clinical comparison

Hi-Rem Post



With Hi-Rem Post, during the removal, the Dentist can clearly feel that the instrument is guided through the post by the macro-fiber, thereby assuring a sense of certainty that it is never felt during the removal of a conventional post.

Conventional Posts



Using conventional posts the results demonstrate that only a partial removal of the post can be undertaken, and that serious damage to the roots occur, such as enlargement of the canal, weakening of the root, and at times, root perforation.

Hi-Rem Post is available in three different shapes

Hi-Rem Polygon Post > oval section > For teeth with non-circular and oval canals

SKRPYOP Hi-Rem Polygon Post Starter Kit 9 Posts (3 for each size 1-2-3) + 3 Drills (1 for each size 1-2-3)

| CODE    | Refill                  | Maxi/Min Coronal Ø | Point Ø | Length | Refill Pack |
|---------|-------------------------|--------------------|---------|--------|-------------|
| RPYOP01 | Hi-Rem Polygon Post # 1 | 2.00/1.20          | 0.80    | 19     | 10 posts    |
| RPYOP02 | Hi-Rem Polygon Post # 2 | 2.20/1.40          | 1.10    | 19     | 10 posts    |
| RPYOP03 | Hi-Rem Polygon Post # 3 | 2.50/1.60          | 1.20    | 19     | 10 posts    |

| CODE  | Refill        | Refill Pack |
|-------|---------------|-------------|
| PYD01 | Polygon Drill | 1 drill     |
| PYD02 | Polygon Drill | 1 drill     |
| PYD03 | Polygon Drill | 1 drill     |

Hi-Rem Prosthetic Post > conical-cylindrical shape > For teeth with no residual or little coronal tissue

NEW SKRPOP Hi-Rem Prosthetic Post Starter Kit 9 Posts (3 for each size 0-1-2) + 3 Drills (1 for each size 0-1-2)

| CODE    | Refill                     | Maximum Ø | Point Ø | Conicity% | Length | Refill Pack |
|---------|----------------------------|-----------|---------|-----------|--------|-------------|
| RPOP100 | Hi-Rem Prosthetic Post # 0 | 1.20      | 0.70    | 0.04      | 19     | 10 posts    |
| RPOP101 | Hi-Rem Prosthetic Post # 1 | 1.40      | 0.80    | 0.05      | 19     | 10 posts    |
| RPOP102 | Hi-Rem Prosthetic Post # 2 | 1.60      | 0.80    | 0.06      | 19     | 10 posts    |
| RPOP103 | Hi-Rem Prosthetic Post # 3 | 1.80      | 0.90    | 0.08      | 19     | 10 posts    |


| CODE  | Refill           | Refill Pack |
|-------|------------------|-------------|
| PD100 | Prosthetic Drill | 1 drill     |
| PD101 | Prosthetic Drill | 1 drill     |
| PD102 | Prosthetic Drill | 1 drill     |
| PD103 | Prosthetic Drill | 1 drill     |

Hi-Rem Endodontic Post > double conical shape > For teeth with moderate quantities of residual coronal tissue

SKREOP Hi-Rem Endodontic Post Starter Kit 9 Posts (3 for each size 1-2-3) + 3 Drills (1 for each size 1-2-3)

| CODE    | Refill                     | Maximum Ø | Point Ø | Length | Refill Pack |
|---------|----------------------------|-----------|---------|--------|-------------|
| REOP201 | Hi-Rem Endodontic Post # 1 | 1.60      | 0.72    | 19     | 10 posts    |
| REOP202 | Hi-Rem Endodontic Post # 2 | 1.80      | 0.88    | 19     | 10 posts    |
| REOP203 | Hi-Rem Endodontic Post # 3 | 2.00      | 1.04    | 19     | 10 posts    |

| CODE   | Refill             | Refill Pack |
|--------|--------------------|-------------|
| HED201 | H-Endodontic Drill | 1 drill     |
| HED202 | H-Endodontic Drill | 1 drill     |
| HED203 | H-Endodontic Drill | 1 drill     |

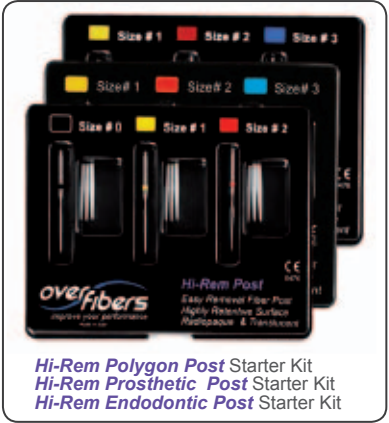

Overfibers S.r.L. - Via Selice 13 - 40027 Mordano (BO) - Italy  
Production & Logistic +39.0542.52153 Fax +39.0532.1920219  
[www.overfibers.com](http://www.overfibers.com) - [info@overfibers.com](mailto:info@overfibers.com)




UK 2013/02 2.500




# Hi-Rem Post

The Only Easy Removal Fiber Post

Hi-Rem Post, the Overfibers' patented fiber post, it is the only one that has the great advantage to be easily removable when it is necessary

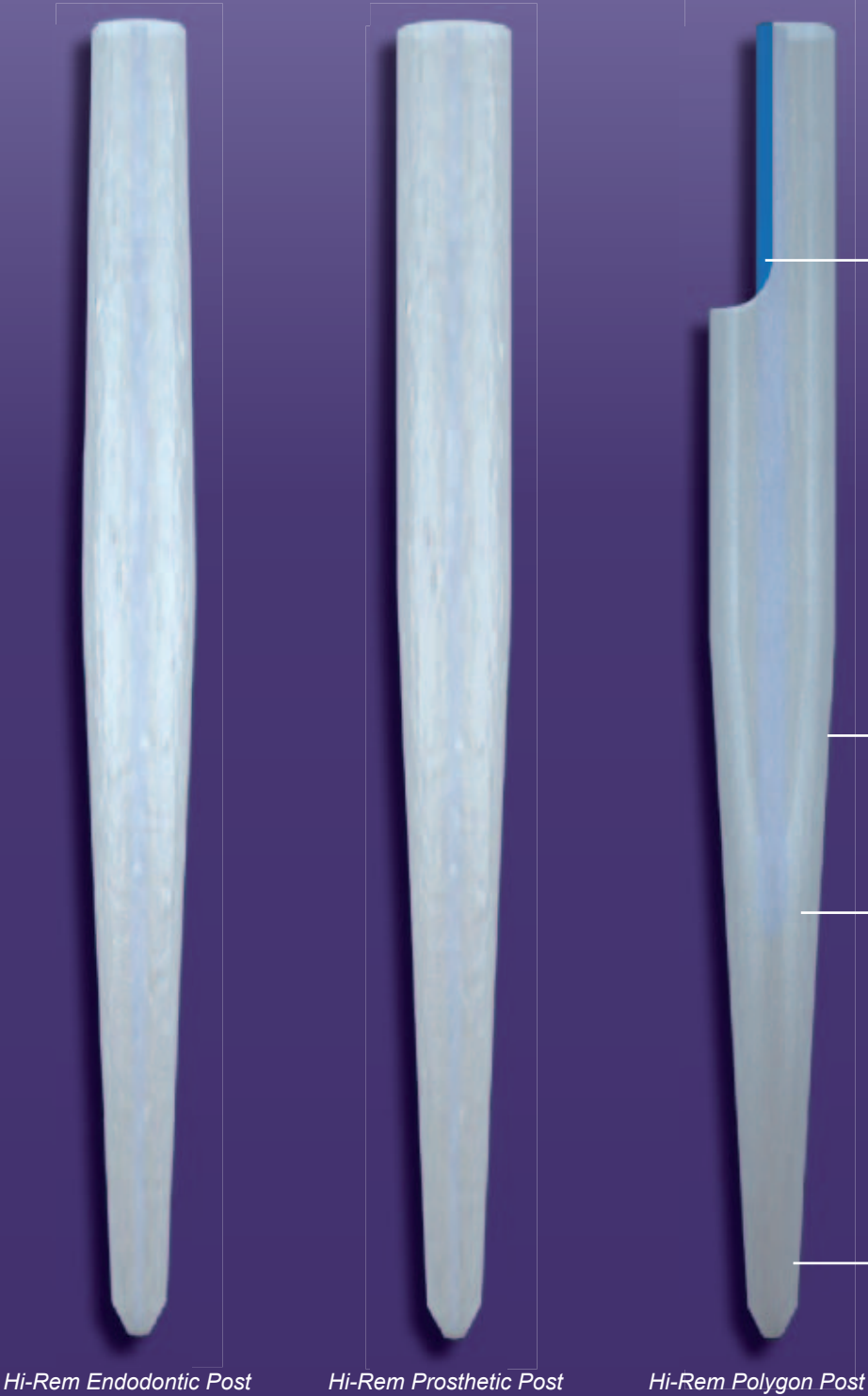

The central longitudinal axis of Hi-Rem Post is made of blue colored soft polymer macro-fiber, placing in this position doesn't affect the resistance of the post (neutral axis).

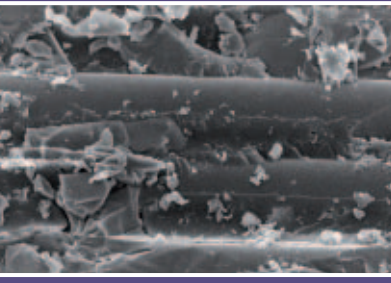

The surface of Hi-Rem Post is extremely rough and retentive in order to maximize the adhesion of the cement. 1000 X

Hi-Rem Post is translucent to accelerate the activation of dual curing cements

Hi-Rem Post is available in three shapes, which have been designed for all clinical requirements

The radiopacity of Hi-Rem Post makes them easily distinguishable from the cement

Today, worldwide dentists prefer translucent and radiopaque fiber posts, which have a notable aesthetic advantage, but they are very difficult to detect inside the tooth when their removal is necessary.

# Hi-Rem Post

The first Easy Removal Fiber Post

In order to make the removal of the post as simple as possible to simplify the daily practice of the dentistry, Overfibers, a company devoted to technological innovation, is pleased to present a Worldwide novelty, Hi-Rem Post, an internationally Patented post.

Created with the very sophisticated Overfibers' "know-how", the main characteristic of the new Hi-Rem Post is that the central longitudinal axis of the post is made of blue colored soft polymer macro-fiber.



## Hi-Rem Post Features and Advantages



|                            |                                                              |
|----------------------------|--------------------------------------------------------------|
| Highly retentive surface   | > increases the post / cement* adhesion                      |
| "S" type glass fibers      | > high flexure strength 1650 MPa                             |
| Balanced flexure E-modulus | > 60 GPa to better stabilize the restoration                 |
| Three shapes               | > for all clinical requirements                              |
| Translucent                | > to accelerate the activation of dual cements               |
| Radiopacity                | > to be clearly distinguishable from the cement              |
| Easy to remove             | > safely with absolute certainty and without tooth damages** |

\* Fiber post bonding using self-etching and resin cements  
S. PALLOTTINI, A. LLUKACEJ, C. MONACO, and P. BALDISSARA, University of Bologna - Italy IADR – Barcelona, 2010 abst. 4493


\*\* Tissue loss during post removal a new post concept evaluation P. BALDISSARA<sup>1</sup>, L.F. VALANDRO<sup>2</sup>, S. BETTAZZONI<sup>1</sup>, and R. SCOTTI<sup>1</sup>, <sup>1</sup>University of Bologna - Italy, <sup>2</sup>Federal University of Santa Maria - Brazil IADR – Barcelona, 2010 abst. 772



It has never been easier to remove a fiber post!



I use Hi-Rem Post with my patients as I prefer to work with absolute certainty.

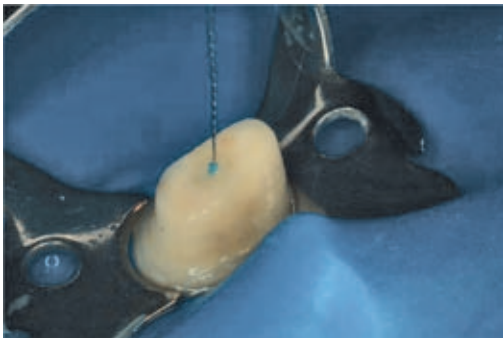


the central longitudinal axis of the post is made of blue colored soft polymer macro-fiber.

Clinical Sequence by video <http://overfibers.com/video.php> "Removal of Hi-Rem Prosthetic Post"  
In only 2 predictable steps, the Dentist removes Hi-Rem Post easily and completely



First step.  
Gutta-percha is safely reached in 30" !

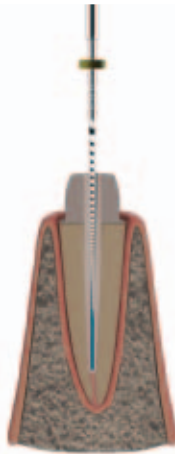

Using a medium-high speed, a nickel-titanium drill of 0.3 mm is able to penetrate the blue macro-fiber. Since the macro-fiber is very soft, the instrument is guided towards the post tip by the hard walls of the post, without ever deviating from the central axis as occurs with normal posts. Gutta-percha can be safely reached in less than 30seconds.



Second step.  
The dentist clears out the interior of the Hi-Rem Post in 90" !



With a Gates-Glidden drill or a Largo/Peeso reamer, the dentist clears out the interior of the Hi-Rem Post, the drill final size depends on the needs of the clinical case.



It has never been easier and faster to remove a fiber post!



At this point, the removal of the post from the canal space can be said to be finished; rapidly, safely, and without sacrificing healthy dental tissue. When the pathological problem is solved, a new dental reconstruction can be undertaken, with the certainty only what was necessary has been removed in order to insert a new Post, a Hi-Rem Post, naturally!

Fiber Post Removal: Comparative Study Using a New Post Concept P. BALDISSARA<sup>1</sup>, D. MELILLI<sup>2</sup>, L. CIOCCA<sup>1</sup>, and L.F. VALANDRO<sup>3</sup>,  
<sup>1</sup>University of Bologna - Bologna Italy, <sup>2</sup>University of Palermo - Palermo Italy, <sup>3</sup>Federal University of Santa Maria - Santa Maria Brazil IADR – Miami, 2009 abst. 183